
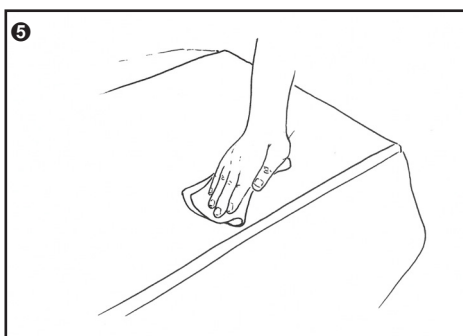
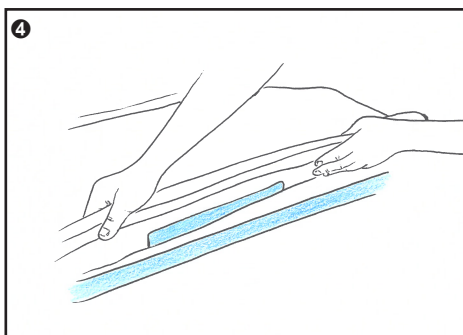
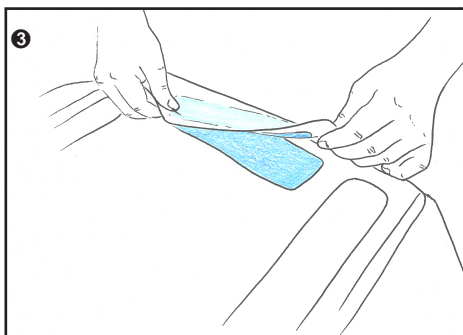
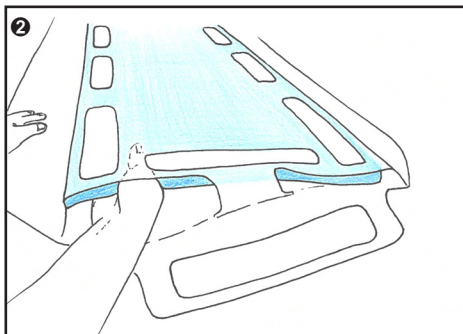
CT Table Cover

Instructions for Use
Gebrauchsanweisung
Mode d'emploi
Istruzioni per l'uso
使用説明書
Instrucciones de uso
Instruções de utilização
Brugsanvisning
Käyttöohje
Bruksanvisning
Användningsinstruktioner
Gebruiksaanwijzing
Инструкция по эксплуатации
Instrukcja obsługi
사용 설명서
使用説明
Návod k použití
تعليمات الاستخدام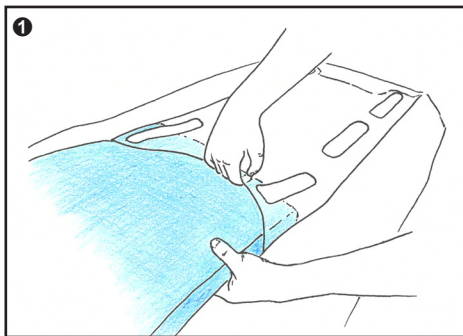

Protection for CT table and table mat

Short-term patient positioning and stabilization device during invasive and non-invasive procedures to be used on intact patient's skin. Non-Sterile and not to be used in the sterile field.

Installation

Remove table mat from CT Table. Clean and disinfect both CT Table as well as Table Mat.

1/2 First, slide head of Table Mat into the head of the CT TABLE COVER (with edge protection) and pull to the very end of the CT TABLE COVER. Fold/bend the rest of the Table Mat to fit it into the foot end of the CT TABLE COVER and pull to the very end.

3 Smoothen Out Table Mat and CT TABLE COVER and close the openings on both ends by using the according flaps.

4 Turn CT TABLE COVER around and attach it to CT Table by using according hook and loop points.

Application

5 Clean the CT TABLE COVER after every exam using any commonly available hospital disinfectant.

Use additional hygienic protection between patient and CT TABLE COVER for every application.

Contraindications: Do not use the product for purposes other than those described.

 Non sterile.

 Do not sterilize. Do not allow direct contact with the skin. Use single-use sanitary cover. Do not use in case of acute open wounds. Edge protection of CT TABLE COVER needs to be placed towards gantry.

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