
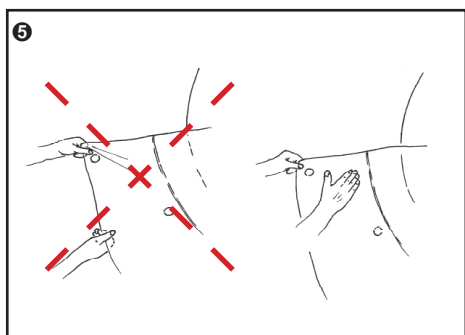
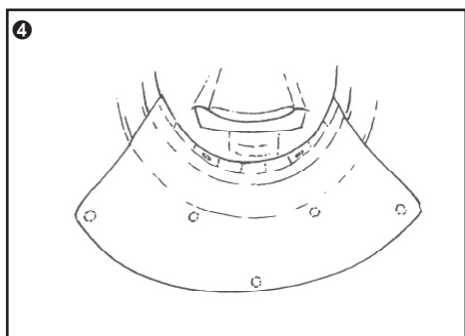
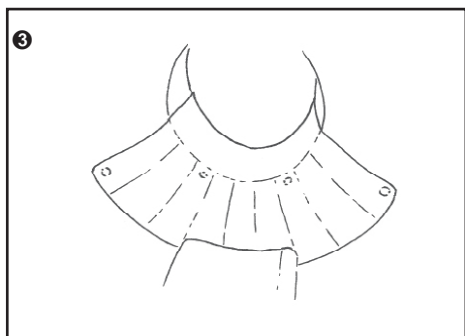
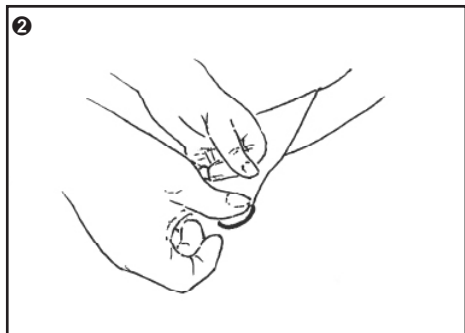
CT Gantry Cover

Instructions for Use
Gebrauchsanweisung
Mode d'emploi
Istruzioni per l'uso
使用説明書
Instrucciones de uso
Instruções de utilização
Brugsanvisning
Käyttöohje
Bruksanvisning
Användningsinstruktioner
Gebruiksaanwijzing
Инструкция по эксплуатации
Instrukcja obsługi
사용 설명서
使用説明
Návod k použití
تعليمات الاستخدام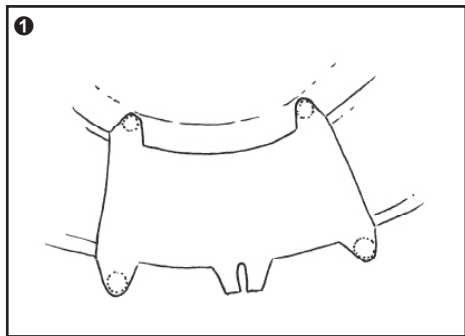

CT Gantry Cover

Reusable hygienic CT gantry protection during invasive and non-invasive procedures to keep the CT gantry protected from penetrating liquids. Covers the lower half of the CT gantry.

May only be used by medical personnel. Non-sterile and not to be used in the sterile field.

Installation

- 1/2 First place the cardboard installation aid in the gantry according to the marking and attach the first four hook and loop patches.
- 3 Place the CT Gantry Cover in the gantry. Use the "Table Side" label to align it correctly within the gantry.
- 4 Attach the CT Gantry Cover to the four pre-installed hook and loop patches. Then attach the other hook and loop patches to the gantry one by one.
- 5 Note: Do not attach remaining hook and loop patches under strong tension of the CT Gantry Cover. To position, simply smoothen the CT Gantry Cover with one hand.

CT Gantry Cover must not cover the lasers of the CT.

CT Gantry Cover must not touch the patient table.

Application

Clean the CT Gantry Cover after every exam using any commonly available hospital disinfectant.

If necessary, remove CT Gantry Cover from gantry for cleaning. To reinstall, start with step "2".

Contraindications: Do not use the product for purposes other than those described.

 Non sterile

 Do not sterilize

Warnings: Dispose of any product that has a tear or appears damaged. Remove the product from the device before calibration.

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